Steam generator

Steam generator that works at 6 bars and need for connection to the water mains.

In addition to the gun with water, steam and water-steam mixture, it is equipped with a fixed pipe for saturated steam outlet. Practical, solid and compact, this device is the ideal ally for dental laboratories.

PILOT LIGHTS

- The light on the left indicates that the heating element is working
- The central light indicates the intervention of the thermostat
- The light on the right indicates that the electric pump is working

PRESSURE GAUGE

It indicates the pressure in the boiler

ON/OFF LIGHT SWITCH

It allows the device to be switched on and off. The green light indicates that the device is in operation

FIXED NOZZLE

For saturated steam delivery. it is activated through the top left button or the pedal



GUN CONTROL BUTTONS

- The left button activated the gun steam
- The right button activates the gun water
- Pressing both switches allows the outlet of water-steam mixture from the gun

GUN

- With safety lock
- Can be positioned on either side L/R
- Ergonomic
- Special heat-resistant material